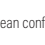
SYMBOLS DESCRIPTION

Full explanation of symbols used on Kerr packaging located at: <http://www.kerrdental.com/symbols- glossary>

REF	Catalogue number
	Manufacturer
LOT	Batch Code
	European conformity
	Consult instructions for use
	Caution
EC REP	Authorized representative in the European Community
	Use-by date
	Do not re-use
	Keep away from sunlight
MD	1. Medical Device
	2. Temperature Storage Limitations: Low 2°C/36°F and High 28°C/82°F

CONTENTS

	3. Unidose Tip
	4. Syringe

en - ENGLISH

OptiShade™

Nanohybrid Universal Restorative Composite ISO 4049- Polymer-Based Restorative Material, Class 2/Group 1

GENERAL INFORMATION

The restorative is available in three composite shades (Light, Medium, Dark) and two special shades (Bleach White and Universal Opaque). All shades are radiopaque with about 250% Aluminum.

This product is intended to be used by qualified dental professionals on a general patient population.

PRODUCT COMPOSITION

100% Aluminum has a radio-opacity equivalent to that of dentin and 200% Aluminum is equivalent to enamel.

The restorative is available in three composite shades (Light, Medium, Dark) and two special shades (Bleach White and Universal Opaque). All shades are radiopaque with about 250% Aluminum.

The fillers comprise selection of chemically infused mixed oxides, pre-polymerized filler bearing glass filler, silica, and ytterbium fluoride (about 81.5% by weight or 651% by volume for the Light, Medium, Da, Dark, and Universal Opaque shades, about 78.5% by weight or 60% by volume for the Bleach White shade). The inorganic fillers is up to 75% by weight or 65% by volume. The Light, Medium, Dark, and Universal Opaque shades have a range particle size of 40 nm to 6 microns. The Bleach White shade has a range particle size of 40 nm to 30 microns. OptiShade contains bis-EMA, Bis-GMA, and TEGDMA resins. None of the ingredients in the product contain pharmacologically active components.

INDICATIONS FOR USE

OptiShade, a nanohybrid dental restorative, is indicated for direct placement in all cavity classes in anterior and posterior teeth. Additional indications include: repair of porcelain/composite restorations, and core build-ups.

CONTRAINDICATIONS

None known.

WARNINGS

- Do not use this product on patients with known methacrylate allergies. Uncured methacrylate resin may cause contact dermatitis and damage to the pulp.
- Avoid contact with skin, eyes, and soft tissue. Wash thoroughly with water after contact.
- Get medical attention for eyes.
- Do not take internally.

PRECAUTIONS

Kerr recommends the following precautions when using OptiShade or any other composite material:

- Provide the patient with protective eyewear.
- Use a rubber dam to isolate the oral cavity.
- Avoid fluid contamination during placement.
- Use of protective gloves and a no-touch clinical technique is strongly recommended. If the composite is touched with commonly used gloves, acrylics may penetrate the glove. Dispose of glove immediately and wash hands.
- Protect the material from ambient light that may initiate polymerization.
- The Unidose tips are designed for SINGLE PATIENT USE ONLY to prevent cross-contamination between patients.

Kerr, OptiShade, and Unidose are trademarks owned by Kerr Corporation.

ADVERSE EVENTS

If a serious incident occurs with this medical device, report it to the manufacturer and to the competent authority for that country in which the user and/or patient is established.

STEP-BY-STEP INSTRUCTIONS

- Wear personal protective equipment when using and handling this product.
- Do not use this product near eugenol-containing products.
- Clean the tooth with non-fluoridated paste and water to remove any debris or stain that may interfere with the shade selection process.
- Match the desired shade based on the chart listed below. A composite mock-up or VITA®-matched shade guide may be used to check the color of the tooth to be restored and any adjacent teeth. In many cases, a single shade of OptiShade will be sufficient to restore the tooth. The shade of the restorative material is affected by the thickness of the composite. Shade selection and mock-ups should match the thickness of the tooth to be restored.

Composite Shades	VITA Shade Guide Equivalent
Light - LT	A1, B1, B2, C1, D2
Medium - MD	A2, A3, C2, D3, D4
Dark - DK	A3.5, A4, B3, B4, C3, C4
Accessory Shades	
Bleach White - BW	Bleach white
Universal Opaque - UO	Blocking discoloration

- After shade selection, isolate the tooth with your preferred isolation method.
- Prepare the tooth using your preferred rotary instruments and using contemporary preparation designs. OptiShade offers excellent blending (chameleon effect) which may be further enhanced by the use of small bevels on preparation margins. If replacing a composite, be sure to remove all existing restorative material from the operative site.
- If a pulp exposure has occurred, place a small amount of calcium hydroxide followed by the placement of a conventional glass ionomer base.
- OptiShade is compatible with all methacrylate dental adhesive systems, such as a product from the KaVo Kerr OptiBond Family. Complete the bonding procedure per manufacturer's instructions.
- If desired, place a liner using a low viscosity flowable composite and light cure according to manufacturer's instructions.
- In situations of discoloration of the tooth structure or repair of porcelain/composite restorations, the Universal Opaque shade can be used to block out the discoloration or hide a metal portion if it exists.

- OptiShade features optimal, non-sticky handling. It permits the use of hand instruments to define most of the restoration anatomy, which will help minimize the time and effort associated with rotary instrumentation.

For Unidose delivery: Inject composite material from the Unidose tip directly into the cavity, starting with the deepest portion.

- For Syringe delivery:** After extruding material onto a pad, back off syringe at least two full turns to relieve pressure on the material.
- Apply composite layers up to 2 mm in thickness each layer for all shades except for Universal Opaque and up to 15 mm in thickness each layer for Universal Opaque, taking care to properly adapt the material to the cavity surface.
- Cure each layer*. Refer to the table at the end of these instructions for recommended curing times of the various shades at various light intensities.
- Adjust occlusion as needed. Complete the finishing and polishing using preferred polishing system.

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*RECOMMENDED CURING TIMES

Adequate curing is essential for restorative function. Uncured or undercured composite material is far more likely to experience some form of clinical failure. Kerr recommends regular inspections of your curing lights to ensure they are functioning and providing the expected irradiance.

OptiShade is intended to be cured by a halogen or LED light with a minimum light intensity of 650 mW/cm² and a light wavelength equal within the 400-520 nm range. Please refer to the table below for curing recommendations. All times refer to a single occlusal surface cure. In any anterior restoration, Kerr recommends additional 10 second cures on both the buccal and lingual surfaces of the tooth.

	Universal Opaque	Dark Shade	Light, Medium, and Bleach White Shades
Incremental Depth	15 mm	2.0 mm	2.0 mm
OptiLux™ (or light with output 650 – 1000 mW/cm²)	40 Seconds	40 Seconds	20 Seconds
Demi™ Ultra / Demi™ Plus (or light with output >1000 mW/cm²)	20 Seconds	20 Seconds	10 Seconds

For all other lights, see manufacturer's recommendation.

CLEANING AND DISINFECTION

Unidose Tips: Do not re-use Unidose tips. **Unidose Light Guns:** Refer to the reprocessing instructions for the manufacturer of the tip gun you are using. **Composite syringes:** To prevent syringing from getting contaminated with saliva or other body fluids, use protective barriers. Always handle the syringe with clean/disinfected gloves. Do not reuse if it becomes contaminated.

Replace the syringe cap after each use to prevent the resin from hardening in the syringe.

STORAGE AND DISPOSAL

Product should be stored at temperature between 2°C to 28°C (36°F to 82°F). If product is stored in a refrigerator allow the product to reach room temperature prior to use. Every effort should be made to protect the composite from visible light prior to use.

The expiration date is marked on packaging. Do not use restorative after expiration date.

For proper disposal always follow Federal or State laws applicable to that country.

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